


Visible Staple Counter for constant indication of staple supply and for efficient use.

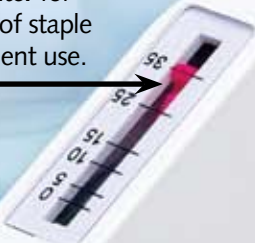

**REFLEX® TL**

Arrow Guide for accurate and consistent placement.



Open Nose Piece and Low Profile for clear view of staple placement.



**REFLEX® ONE**

# CONMED REFLEX® SKIN STAPLERS

**Unequalled Quality, Unequalled Control**

- Two stapler styles — Reflex® One and Reflex® TL
- Two stapler sizes — Regular and Wide
- Confirmation of complete staple formation
- Excellent visibility
- Visible staple counter for more efficient use




**REFLEX® EXTRACTOR**

Ergonomically Designed to help reduce patient discomfort.




# CONMED Reflex® Skin Staplers

Delivering The Performance You Demand!

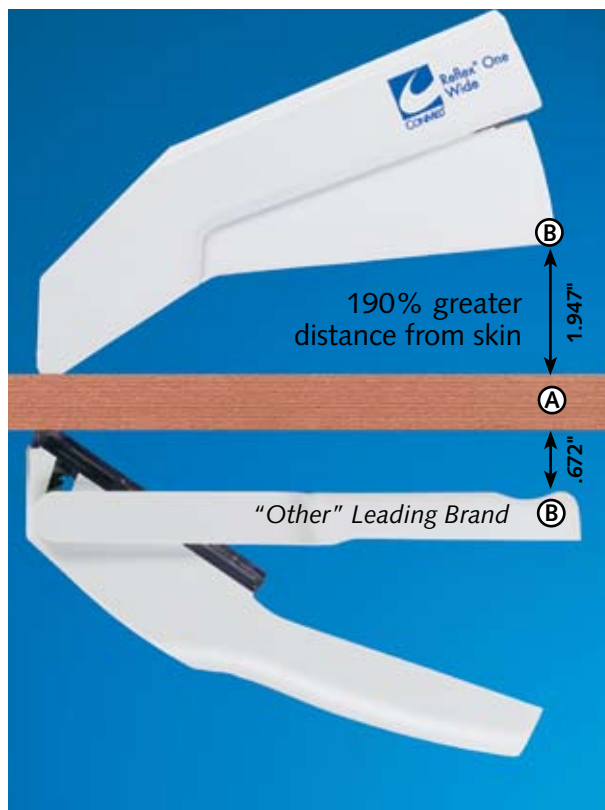

CONMED's Reflex® One and Reflex® TL ergonomic design gives you more area between the handle and the skin (B>A). This keeps the nose of the stapler parallel to the skin allowing proper staple formation, good tissue proximation, and less chance of the onset of edema. When edema occurs, tissue grows over the staple, making for difficult and painful staple extraction.

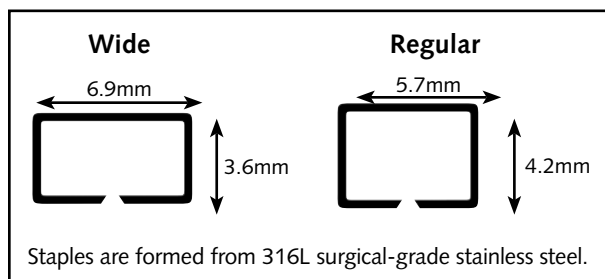

## Reflex® One Skin Stapler

- Flexibility of movement.
- Fully Retracting Anvil permits staple release while moving forward or backward unlike competitive models which require you to "back off" before moving forward.
- Reflex® One releases the staple when the handle is fully depressed. Complete staple formation is confirmed by Audible and Tactile Feedback.

## Reflex® TL Tissue Lift Skin Stapler

- Additional control.
- The Reflex® TL incorporates a Semi Retracting Anvil that only releases a staple when the handle is fully released.
- This Semi Retracting Anvil gives the TL its "Tissue Lift" which assists in approximating the incision for easier placement of the next staple.

## Excellent Visibility

- Constant, visible indication of available staple count promotes more efficient use of supplies.
- Arrow guide for accurate placement.
- Open Nose Piece and low profile allow clear view of staple placement.

## Superior Ergonomics

- Ergonomic design that helps prevent staple embedding and staves off the onset of Edema.
- Reflex®'s shape and balance fit any hand size for better feel and smoother action.

### ORDERING INFORMATION

Product Description	Packaging	Reference #
<b>Reflex® One Skin Stapler — Wide</b>		
35 Wide	6/box	8535
25 Wide	6/box	8525
15 Wide	6/box	8515
<b>Reflex® One Skin Stapler — Regular</b>		
35 Regular	6/box	8035
25 Regular	6/box	8025
15 Regular	6/box	8015
<b>Reflex® TL Skin Stapler with Tissue Lift — Regular</b>		
35 Regular	6/box	8635
25 Regular	6/box	8625
15 Regular	6/box	8615
<b>Reflex® TL Skin Stapler with Tissue Lift — Wide</b>		
35 Wide	6/box	8735
25 Wide	6/box	8725
15 Wide	6/box	8715
Reflex® Extractor — Wide/Regular	12/box	3031
CONMED "Value" Extractor — Wide/Regular	12/box	3034

#### CONMED CORPORATION PRODUCT AREAS:

ARTHROSCOPY • ELECTROSURGERY • ENDOSCOPY • ENDOSURGERY • GASTROENTEROLOGY • INTEGRATED SYSTEMS • PATIENT CARE • POWERED INSTRUMENTS • PULMONOLOGY



525 French Road  
 Utica, NY 13502  
 (315) 797-8375  
 Customer Service: 1-800-448-6506  
 FAX: 1-800-438-3051  
 International Sales: +1(315) 797-8375 • FAX: +1(315) 735-6235  
 email: info@mail.conmed.com  
 www.conmed.com




# Reflex<sup>®</sup> Skin Staplers — Procedure for Clinical Use


## Reflex<sup>®</sup> One Skin Stapler




Reflex<sup>®</sup> One with Regular or Wide Staples fully releases staple from handle immediately upon staple formation.

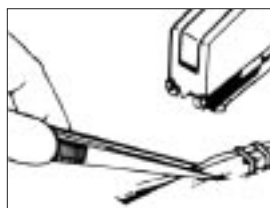


- Approximate skin with a forceps or similar device.
- Pull "up" on skin to "evert" the tissue.

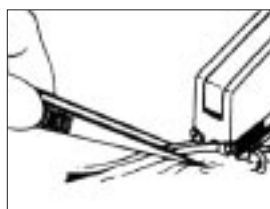

- Position stapler lightly on the everted skin tissue.
- Touch the skin lightly to allow the staple to float above the incision.
- This technique results in better cosmetic closure and easier staple removal.

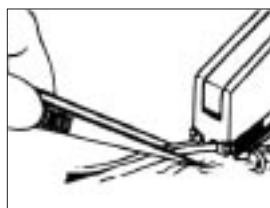

- Squeeze stapler handle fully.
- Both an audible "click" will be heard and a tactile response will be felt when the staple is fully formed.

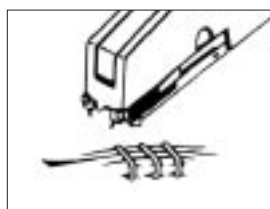

- The staple is released immediately from the stapler upon formation.
- The Stapler handle can be removed from the incision in any direction.


## Reflex<sup>®</sup> TL Skin Stapler



Reflex<sup>®</sup> TL (Tissue Lift) with Regular or Wide Staples releases staple from handle after staple formation, but only after user fully releases handle.



- Approximate skin with a forceps or similar device.
- Pull "up" on skin, "everting" the tissue.

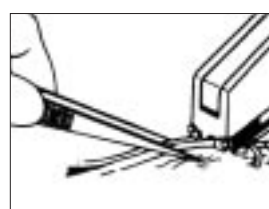


- Position stapler lightly on the everted skin tissue.
- Touch the skin lightly to allow the staple to float above the incision.
- This technique results in better cosmetic closure and easier staple removal.



- Squeeze stapler handle fully.
- A tactile response will be felt when the staple is fully formed.




- With handle still in the closed position, the staple is fully formed but it will not release.
- TISSUE LIFT lets the user "pull up" on the skin helping the user approximate and evert the skin.
- This better facilitates the next staple placement.

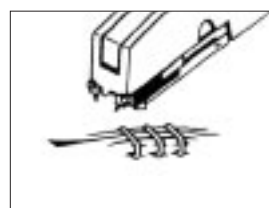

- Staples are released from the handle only after the handle is in the fully open position.



English

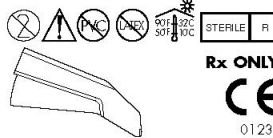

**Reflex® One and Reflex® TL Single-Use Skin Stapler**

**Instructions for Use**

**Description**

Staplers are available in quantities of 35, 25 and 15 stainless steel staples in wide and regular widths.

**Formed Staple Dimensions**

Regular staples .51mm diameter wire.



Wide staples .56mm diameter wire.



**Indications**


The Reflex®One and Reflex®TL Skin Staplers are used for closure of skin in a wide variety of surgical procedures.

**Contraindications**

1. When it is not possible to maintain at least 5mm distance from the stapled skin to underlying bones, vessels, and internal organs, the use of staples for skin closure is contraindicated.

**Directions for Use**

1. Evert and approximate skin tissue.
2. Position stapler lightly on everted skin tissue. Touching the skin lightly will allow the stapler to float above the incision, resulting in better cosmetic closure and easier removal.

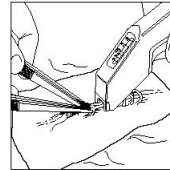

**Reflex®One:**

- 3a. Squeeze trigger fully and then release trigger fully. The staple will release automatically from the anvil with trigger release. The stapler can be removed from the incision in any direction.

**Reflex®TL:**

- 3b. Squeeze trigger fully and then release trigger fully. Back the instrument off the staple to remove the anvil out from under the staple.
4. Move stapler to next desired position and repeat procedure.
  5. Quantity of staples remaining is shown in the counter window.

**Cautions**

1. This device is provided sterile for single-use only. Do not use if package is opened or damaged. Do not resterilize. Discard after use.
2. Always ensure trigger is fully closed and released to properly feed the next staple into position.

U.S. Patent Number 4,558,810



Utica, NY 13502  
(315) 797-8375 • FAX (315) 797-0321  
Customer Service (USA) 1-800-448-6506  
e-mail: info@conmed.com

© CONMED Corporation, Printed in U.S.A.

European Authorized Representative  
MDSS GmbH  
Burckhardstr 1  
D - 30163 Hanover  
Germany

Made in U.S.A.  
For International orders or inquiries,  
please contact CONMED International Sales  
(315) 797-8375 • FAX (315) 735-6235  
www.conmed.com

P/N 10299 Rev A 10/01

ARTHROSCOPY • ELECTROSURGERY • ENDOSURGERY • IMAGING • INTEGRATED SYSTEMS • PATIENT CARE • POWERED INSTRUMENTS



525 French Road  
Utica, NY 13502  
(315) 797-8375  
Customer Service: 1-800-448-6506  
FAX: 1-800-438-3051  
International Sales: +1(315) 797-8375 • FAX: +1(315) 735-6235  
email: info@mail.conmed.com  
www.conmed.com

## ConMed Reflex® Skin Staplers

### Features:

- 15, 25, and 35 staple count models
- “Best In Class” ergonomics
- Audible and tactile feedback
- Visible counter
- Reflex® One Stapler with retracting anvil or
- Reflex® TL “Tissue Lift” Stapler

### Benefits:

- Choice
- Helps prevent embedded staples
- Confirms staple formation
- Speed
- Lets staple release immediately from stapler
- The TL Stapler holds on to the staple until staple release is desired assisting in approximating skin

### Reflex® One Skin Stapler, 15 Staples

	Ref. No.
Regular, 1/pkg, 6/cs .....	8015
Wide, 1/pkg, 6/cs .....	8515

### Reflex® One Skin Stapler, 25 Staples

Regular, 1/pkg, 6/cs .....	8025
Wide, 1/pkg, 6/cs .....	8525

### Reflex® One Skin Stapler, 35 Staples (Our most popular model!)

Regular, 1/pkg, 6/cs .....	8035
Wide, 1/pkg, 6/cs .....	8535

### Reflex® TL “Tissue Lift” Skin Stapler, 15 Staples

Regular, 1/pkg, 6/cs .....	8615
Wide, 1/pkg, 6/cs .....	8715

### Reflex® TL “Tissue Lift” Skin Stapler, 25 Staples

Regular, 1/pkg, 6/cs .....	8625
Wide, 1/pkg, 6/cs .....	8725

### Reflex® TL “Tissue Lift” Skin Stapler, 35 Staples

Regular, 1/pkg, 6/cs .....	8635
Wide, 1/pkg, 6/cs .....	8735

### Reflex® Extractor

Wide/Regular, 1/pkg, 12/cs .....	3031
----------------------------------	------



Reflex® One



Reflex® One Wide



Reflex® TL



Reflex® Extractor